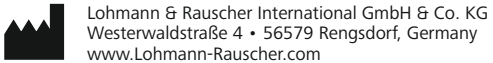
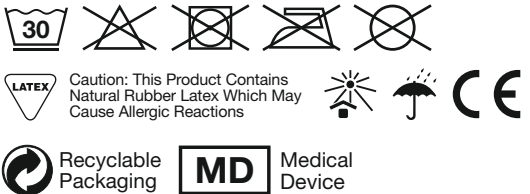
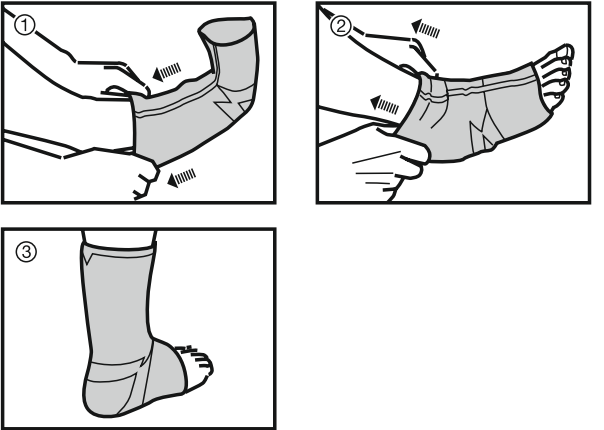
Achillessehnenbandage
Bandage pour tendon d'Achille
Support for the Achilles tendon
Soporte para el tendón de Aquiles
Bendaggio per il tendine d'Achille
Achillespeesbandage

4041549/2020-11

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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:

Cellacare® Achillo is a knitted support with a compression effect for the region of the ankle joint and the Achilles tendon. An integrated, anatomically shaped silicone insert encloses the tendon canal from heel to lower leg. The support can be worn on the right or left foot.

Product composition:

Cotton, elastane, elastodiene (natural rubber latex), polyamide, polyurethane, silicone

Intended purpose:

Support for compression of the Achilles tendon

Indications:

- Acute, chronic or posttraumatic irritation in the Achilles tendon area
- Peritendinitis, peritendinosis, tendinitis, tendinosis
- Bursitis at the Achilles tendon
- Postoperative treatment after surgeries on the Achilles tendon / in the Achilles tendon area

Contra-indications:

- Known allergy and / or hypersensitivity to any of the product components

Side effects:

In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Achillo unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Achillo shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before application of the product.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of circulatory and / or sensory disorders (e.g. with diabetes mellitus); arterial diseases (occlusive arterial disease); venous disease (phlebitis, venous thrombosis), impaired lymph circulation in the legs please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- Cellacare® Achillo must be removed during longer periods of rest (e.g. when sleeping) to avoid undesirable congestive effects.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.
- Cellacare® Achillo is intended for use on a single patient only.

- The product's durability may be compromised by certain factors, e.g. objects with sharp edges.
- Cellacare® Achillo must not be used beyond the last day of the month indicated on the packaging (expiry date).
- This product contains natural rubber latex, which may cause allergic reactions.
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information:

Usage:

1. Grip the upper end of the support and pull it over the top of the foot. The seam must be centered over the top of the foot (Fig. 1).
2. Grip the back of the support and pull it over the heel (Fig. 2).
3. The silicone insert now encloses the Achilles tendon (Fig. 3). Correct the fit if necessary.
4. One of the enclosed heel wedges can be used for temporary relief of the Achilles tendon on the injured foot. The other heel wedge must then be also worn under the healthy foot's heel in order to avoid pelvic obliquity.

Care recommendations:

Wash the support at 30 °C (delicate cycle) with like colours. Do not bleach. Allow Cellacare® Achillo to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. If possible use a mesh bag.

Storage and transport conditions:

Cellacare® Achillo must be kept away from light and moisture (preferably in the original packaging).

Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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